


MoLab Update

1. Installation of MoLab Update

MoLab Update is the CMA Utility program to update the Coach CE program on MoLab. At the CMA website there are two installation files available, MoLabUpdate_6_xx.exe for 32-bits Windows and MoLabUpdate_6_xx_x64.exe for 64-bits Windows (where xx indicates the update version number, the first available version is version 6.41).

- Download the latest MoLab Update installation file via the CMA website.
- Double-click the downloaded file.
- The MoLab Update Installer starts.


- Select the language for MoLab Update and click **OK**.

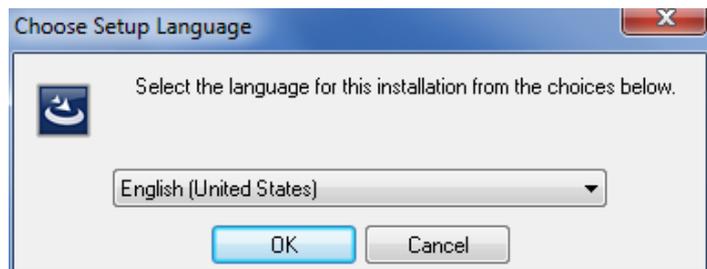


- The installer prepares your computer for installation.




- Click **Continue** in the **Welcome** dialog.

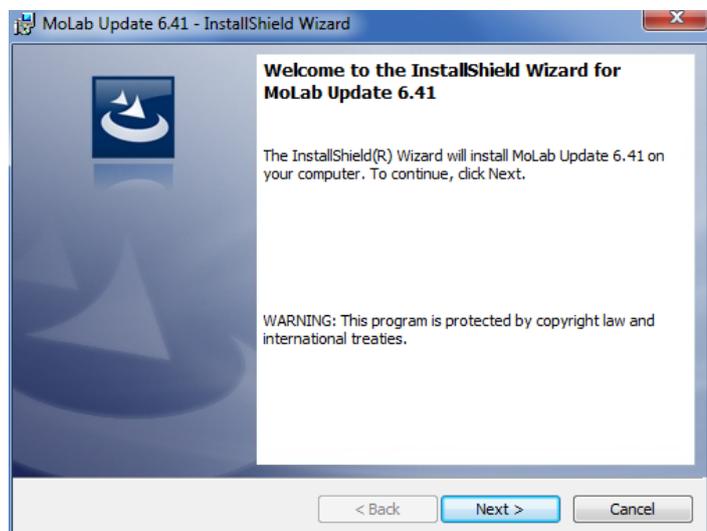


- Click **Install** in the **Ready to Install the Program** window.

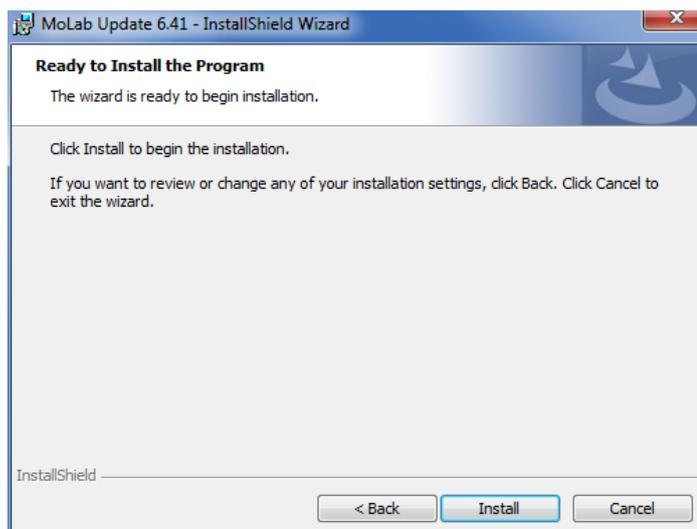

- The process of installation starts.

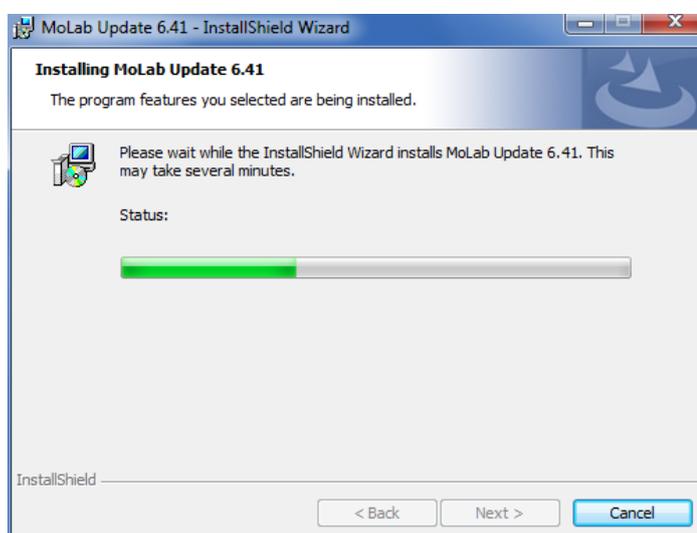

- Click **Finish** to complete installation.
- The MoLab View icon is added to the group **CMA Utilities**.

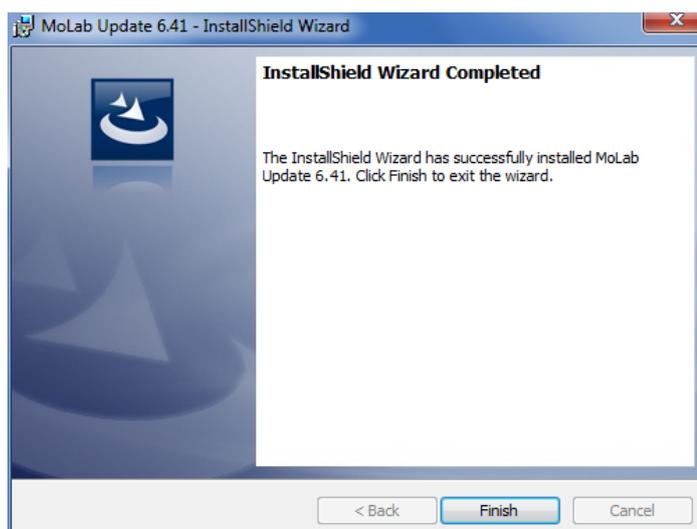


2. Updating Coach CE on MoLab

For using MoLab Update, MoLab has to be:

- connected to the host PC via a USB port,
- Windows CE Services software, Microsoft ActiveSync or Windows Mobile Device Center, has to be installed on the PC¹, and
- the Coach 6 PC program should be closed.

1. Start MoLab View:
 - Click on the Windows **Start** button and select **Programs**.
 - Then go to **CMA Utilities** and select **MoLab Update**.
2. The MoLab Update program starts.
3. Carefully read the given information.

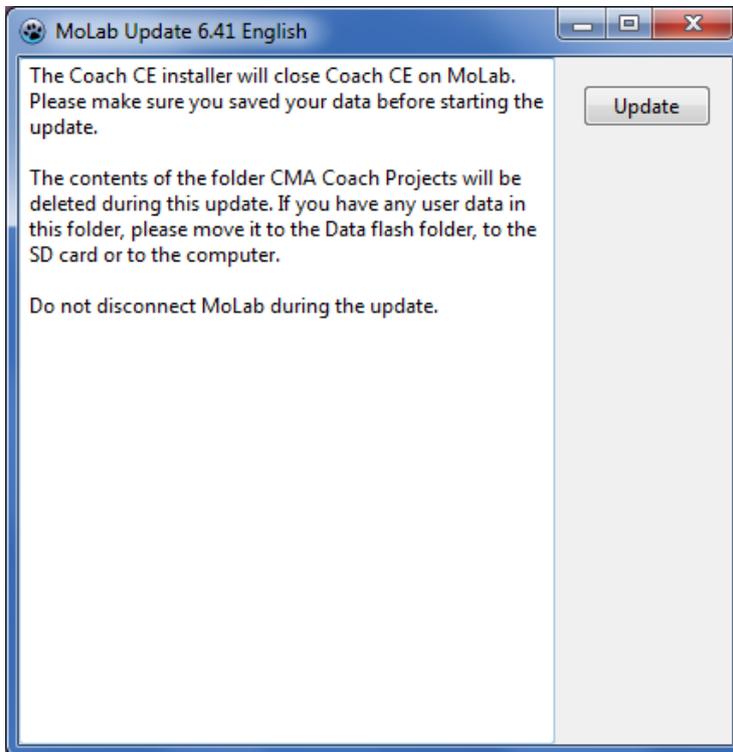

4. When you are ready click the **Update** button to update Coach CE on MoLab.



CENTRE FOR MICROCOMPUTER APPLICATIONS

<http://www.cma-science.nl>

¹ Notice that in the Mobile Device Center software you should use the option 'Connect without setting your device'.